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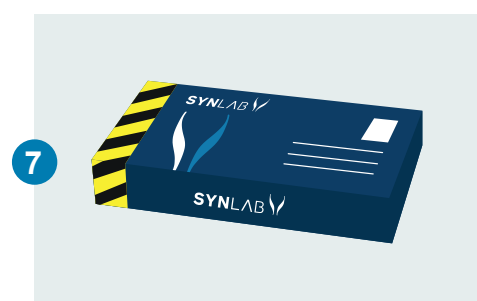
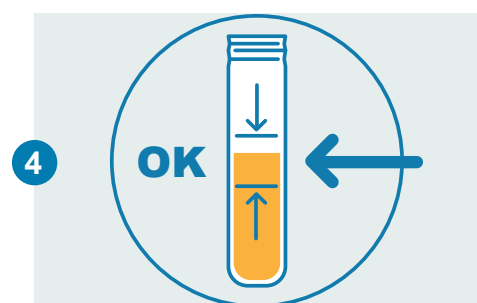
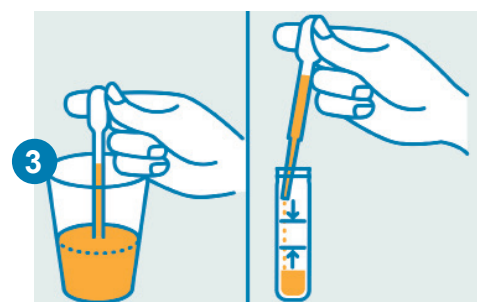
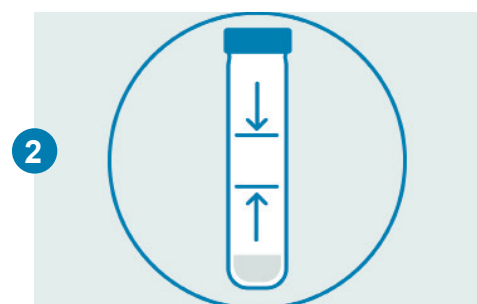
CHLAMYDIA AND GONORRHEA TEST, MEN

PLEASE PAY ATTENTION TO THE FOLLOWING REQUIREMENTS FOR SELF-SAMPLING

- Perform the test at least 5 days after having unprotected sex.
- Do not wash your genitals before taking the sample.
- The test tube contains a liquid that you should not spill. If you do, you must order a new test. Do not touch the test tube's foil sealed cap and make sure it does not break. If you do, you must order a new test.
- Note! If you spill liquid on your skin, you must immediately rinse it with water and soap. If liquid gets into your eyes, rinse it immediately with water. If irritation occurs, please contact medical care.

TAKING THE SAMPLE

1. Take out the disposable plastic cup found in the box. Press in the sides to return it into its original shape. Urinate and collect the first part of the urine stream into the cup. Half a cup is enough. Place the cup with urine on a steady surface.
2. Open a screw-capped sample tube in the package. Make sure that the preservative solution is on the bottom of the tube. Preservative solution shouldn't be discarded.
3. Take the disposable pipette out of a plastic packaging. Use it to transfer urine from the cup into the sample tube. Draw some urine into the pipette by pressing the thick end and then releasing the pressure when the tip of the pipette is immersed in the urine.
4. Stop when the surface reaches the area between the black bars. Pour the rest of the urine into the toilet and discard the plastic cup.
5. To prevent the urine from leaking out during transportation, screw in the cap of the sample tube firmly. Turn the tube up and down 3–4 times to mix the urine with the preservative.
6. Put the tube into the resealable plastic bag. Do not remove the small piece of filter paper inside the bag. Seal the bag and put it into the box.
7. Close the box carefully by inserting the wing-lets into the slots in the corners. Send the box in via regular mail. The postage is prepaid and no stamp is needed.

CHECKING THE ANALYSIS RESULT

Once your sample has been analysed, you'll receive an e-mail. You can check your results from testadigemma.se/results after you have logged in with your bank ID.